# Lembar Disposisi

Surat diterima tanggal : 14 Agustus 2023

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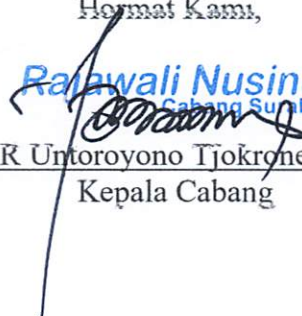
14/8/23

## Service Report

|                 |                                     |                    |                         |                       |
|-----------------|-------------------------------------|--------------------|-------------------------|-----------------------|
| No.<br>02308007 | Inst. Type STEAMER<br>DGM / AND 300 | Serial No.<br>1566 | Engineer<br>DESI YENIAR | Date<br>10-11 AGUS 23 |
|-----------------|-------------------------------------|--------------------|-------------------------|-----------------------|

|  |  |
|--|--|
| Customer's Name<br><b>RS MATA UNDAAN</b> | Address <b>JL. UNDAAN KULON NO. 19 PENELEH<br/>KEL. GENTENG SURABAYA, JATIM.</b> |
|  | Telp. No. -  |

|  |  |                        |
|--|--|------------------------|
| <b>Job type codes</b>  | <b>Charge type codes</b>   | Warranty end date<br>- |
| <input type="checkbox"/> 01 Normal Call<br><input type="checkbox"/> 02 Installation<br><input type="checkbox"/> 03 Pack/move/reinstall<br><input checked="" type="checkbox"/> 04 Maintenance<br><input type="checkbox"/> 05 Customer Training<br><input type="checkbox"/> 06 Modification<br><input type="checkbox"/> 07 Workshop<br><input type="checkbox"/> 08 Application<br><input type="checkbox"/> 09 Others<br><input checked="" type="checkbox"/> 10 TROUBLE SHOTING | <input type="checkbox"/> Warranty<br><input type="checkbox"/> Invoice<br><input type="checkbox"/> Service Contract<br><input type="checkbox"/> Internal, comment |                        |

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|                                      |                          |
|--------------------------------------|--------------------------|
| Customer's Signature & Company Stamp | Engineer's Signature<br> |
|--------------------------------------|--------------------------|

Customer Copy

SERVICE

14/23  
18

**Table 4. Sterilizer maintenance schedule**

| SUBASSEMBLY  | Recommended period |          |         |         |        | COMMENTS  |
|--|--------------------|----------|---------|---------|--------|---|
|  | 6 months           | 3 months | 1 month | 2 weeks | 1 week |   |
| <b>CHAMBER</b>   |                    |          |         |         |        |   |
| insides of a sterilization chamber   |                    |          |         |         | CL     |   |
| external sterilizer walls  |                    |          |         |         | CL     |   |
| door gasket *  | R                  |          | CL      |         | CHK    | <i>check the condition of door gasket (without taking it out), watch out for scratches and deformations, determine if the gasket is covered with a film of grease on its entire surface. Use MOLYKOTE grease on the gasket.</i> |
| drain filter of the chamber  |                    |          |         |         | CL     |   |
| <b>INSTALLATION AND SUBASSEMBLIES</b>  |                    |          |         |         |        |   |
| boiler of a steam generator**  | CL                 |          |         |         |        | <i>to remove scale use non-aggressive formulae</i>  |
| water level electrodes**   |                    |          |         |         |        |   |
| – steam generator  | CHK                |          |         | CL      |        | <i>check the length, adjust</i>   |
| float valve**  | CL                 |          |         |         |        |   |
| water meter glass**  | CL                 |          |         |         |        |   |
| (supplying) water pump **  | CHK                |          |         |         |        |   |
| safety valve   |                    |          | CHK     |         |        |   |
| vacuum pump  | CHK                |          |         |         |        |   |
| check valves   | CL                 |          |         |         |        |   |
| steam trap   |                    |          | CL      |         |        |   |
| sediment traps on entries of steam, water and compressed air supply installations  |                    |          | CL      |         |        | <i>a compressed air filter is equipped with an automatic dehydration system</i>   |
| connection tightness - leaks   |                    |          | CHK     |         |        |   |
| pressure cut-outs  | CHK                |          |         |         |        | <i>settings values</i>  |
| sterile air filter   | R                  |          |         |         | CHK    |   |
| <b>ELECTRIC CONTROL</b>  |                    |          |         |         |        |   |
| cables, electrical connections condition   |                    | CHK      |         |         |        | <i>especially in heating circuits **</i>  |
| <p>where CHK – check, CL – clean, R – replace.</p> <p>* – replace as needed, no more seldom than once in half a year</p> <p>** – concerns versions with integral (built-in) steam generator.</p> |                    |          |         |         |        |   |

**Care and maintenance of electrical equipment.**

Electrical devices should be used in accordance with national or local regulations.

Electrical box must be always closed. During the technical inspection, electrical installation must be disconnected from the mains.